

SIGVARIS



Product Catalogue

(Products not available on prescription)

SIGVARIS

the company

SIGVARIS have over 50 years experience of producing medical compression stockings as a solution for treatment of venous disease and lymphoedema.

Today, over 60 million legs have worn SIGVARIS medical compression stockings worldwide.

As a company we are committed to providing world class products to people suffering from venous disease and lymphoedema and are contributing each day to making SIGVARIS the world leader and expert in medical compression stockings.

We at SIGVARIS are passionate about providing the best products for patients.

Obviously patients have different needs, and these can always be changing.

This is why SIGVARIS have the largest range of medical compression hosiery available.

Contents

Norms & Standards	4
Helpful Tips	5
Caring for your compression garments	7
Compression Levels & Clinical Indications	8
Product Range	9
Grip Top Styles	10 & 11
Unisex products	
Comfort	14
Cotton	16
Traditional	18 & 19
Products for Women	
Attrait	22 & 23
Audace	24 & 25
Bambou	26 & 27
Coton	28 & 29
Diaphane	30 & 31
Intrigue	32 & 33

Magic	34 & 35
Opalis	36 & 37
Soyance	38 & 39
Vertige	40 & 41
Products for Men	
Bambou	44 & 45
Expert	46 & 47
Graphik	48 & 49
Initial	50 & 51
Urban	52 & 53
Wool	54 & 55
More	
Accessories	58
Gloves	59
SIGVARIS ChipSleeve	60 & 61
Traveno Travel socks	62 & 63

Norms and Standards

Below is a useful guide to show the different classes of compression available in this country.

Medical compression stockings

	BS 6612: 1985	AFNOR G 30.102	RAL-GZ 387: 2000
Testing method	BRITISH STANDARD	FRENCH STANDARD	GERMAN STANDARD
CLASS I	14 - 17 mmHg	10 - 15 mmHg	18 - 21 mmHg
CLASS II	18 - 24 mmHg	15 - 20 mmHg	23 - 32 mmHg
CLASS III	25 - 35 mmHg	20 - 36 mmHg	34 - 46 mmHg
CLASS IV	Not reported	> 36 mmHg	> 49 mmHg

There is no European Norm or Standard

Helpful Tips

Check list before ordering

As a reminder you can order by:

- Phone: **01264 326 666** for customer care line.
- Email: customerservices@sigvaris.com
- Fax: 01264 369 950

Here is a quick check list which you may find helpful when ordering.

- Product name e.g. Comfort
- Compression class e.g. RAL class 2
- Size required e.g. small/medium/large
- Garment type e.g. calf/thigh/tights
- Length e.g. short/normal/long
- Colour
- Product code (If known)
- Open toe or closed toe
- Male or female patient (If ordering a product with a grip top)
- Your name, address and contact details.



Getting the most from your compression garments

Ideally the compression garments should be placed in a washing bag and placed on a delicate wash.

Use a suitable washing product: Something like a natural fabric washing powder or liquid, or SIGVARIS washing solution.

Caution: Synthetic washing powders, fabric softeners, fluorescent whitening agents, turpentine and acid or bleach products may damage the elastic yarn in the compression garment.



After washing, use a low spin speed (never wring the product), towel dry and put in an airing cupboard or put on a washing line to dry naturally. Please do not use a tumble dryer to dry your hosiery or place directly onto a radiator.

If looked after, SIGVARIS medical compression garments can last up to 6 months. Factors to consider to make sure wearers get the most from their hosiery:

- How often they are worn and how they are put on and taken off?
- How have the garments been washed?

Be aware of finger nails and rings snagging and laddering the stockings.


Talc, bath and body oils can affect the silicone top bands on stockings so they may not be as 'sticky' as when they were first worn. Cleaning with diluted alcohol or SIGVARIS cleaning wipes remedies this problem.

Stockings should not be worn more than 6 months as they lose their elasticity. Some people say that their new stockings are firmer than their last pair. This is often because they had been wearing the previous products too long and perhaps not getting the best from their hosiery.

Compression Levels & Clinical Indications

Compression Levels	Indications
15-21 mmHg	<ul style="list-style-type: none"> • Heaviness and fatigue in the leg • Mild varicosis during pregnancy • Mild varicosis without significant oedema • Prophylaxis of thrombosis and embolism for immobile patients • Prophylaxis of the Economy-class-syndrome Product examples: Diaphane / Opalis / Soyance / Audace / Vertige / Bambou / Coton / Initial / Urban / Graphik / Comfort / Cotton / Magic / Wool
22-32 mmHg	<ul style="list-style-type: none"> • More severe varicosis during pregnancy • Varicosis with mild oedema • After superficial and deep thrombophlebitis • After sclerotherapy or surgery to maintain therapeutic success • Chronic venous insufficiency • After healed venous ulcer in chronic venous insufficiency • DVT to treat symptoms and oedema to prevent or manage manifestations of the post thrombotic syndrome PTS Product examples: Diaphane / Coton / Expert / Traditional / Cotton / Comfort / Magic
33-46 mmHg	<ul style="list-style-type: none"> • Pronounced varicosis with marked oedema • Advanced chronic venous insufficiency • For healing of serious and recurrent ulcers • Post traumatic oedema • Reversible lymphoedema Product examples: Traditional / Cotton
> 49 mmHg	<ul style="list-style-type: none"> • Severe post thrombotic syndrome • Irreversible lymphoedema Product examples: Traditional

Product Range

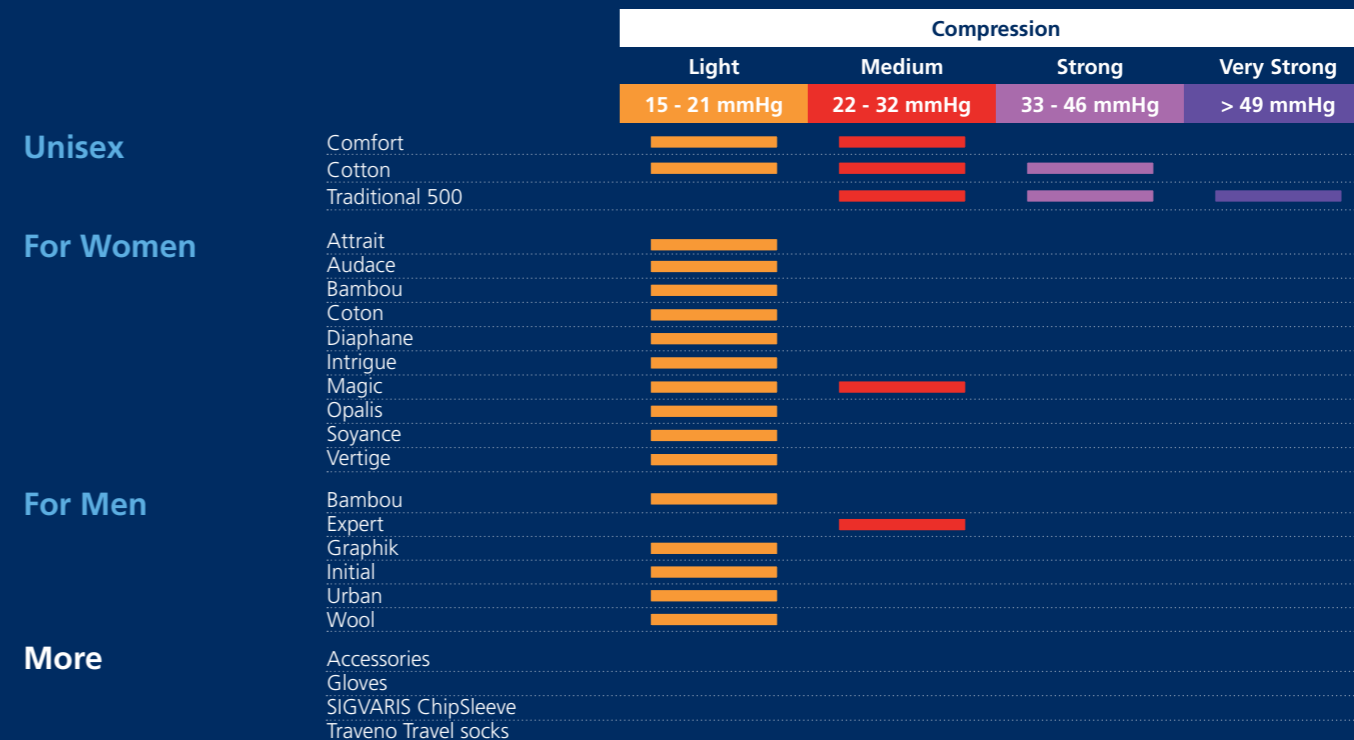

Grip Top Styles

Magic

Sensinova grip top



Cotton

Sensinova grip top



Traditional

Knobbed grip top



Quadrinova knobbed grip top



Quadrinova knobbed grip top



Comfort

Quadrinova knobbed grip top



Attrait

Sensinova grip top



Intrigue

Sensinova grip top



Diaphane

Sensinova grip top



Opalis

Sensinova grip top



Audace & Vertige

Sensinova grip top



Expert & Urban

Sensinova grip top



Unisex Products

COMFORT

For the highest wearing comfort

- Provides the perfect snug fit for all shapes
- Uncomplicated to handle
- Opaque finish
- Conceals venous problem zones
- Available in Closed and Open toe
- Sizes available from XXS - XL PLUS

RAL CI 1 : 18 - 21 mmHg (light compression)
 RAL CI 2 : 22 - 32 mmHg (medium compression)

COMFORT CI 1



- Calf with soft top
- Thigh with soft top
- Thigh with grip top
- Tights
- Maternity tights

■ Grey

COMFORT CI 2

- Calf with soft top
- Thigh with soft top
- Thigh with grip top
- Tights
- Maternity tights

■ Grey

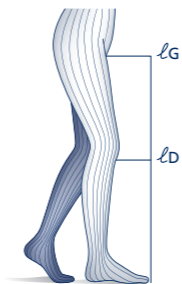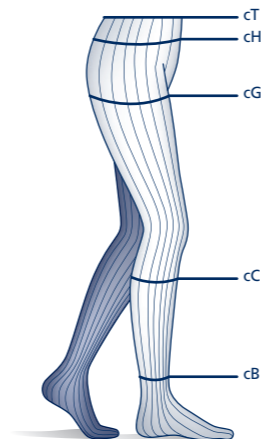

Circumference in cm	xx-small	xx-small PLUS	x-small	x-small PLUS	small	small PLUS	medium	medium PLUS	large	large PLUS	x-large	x-large PLUS
cT	60 - 75	65 - 80	65 - 80	75 - 90	75 - 90	85 - 100	85 - 100	95 - 110	95 - 110	105 - 120	105 - 120	115 - 130
cH	75 - 90	80 - 95	80 - 95	85 - 100	85 - 100	95 - 110	95 - 110	105 - 120	105 - 120	115 - 130	115 - 130	125 - 140
cG	42 - 54	47 - 59	47 - 59	52 - 64	52 - 64	56 - 68	56 - 68	61 - 74	61 - 74	66 - 79	66 - 79	76 - 85
cG Grip top	42 - 52	47 - 56	47 - 56	52 - 61	52 - 61	56 - 65	56 - 65	61 - 71	61 - 71	66 - 76	66 - 76	73 - 85
cC	25 - 32	29 - 36	29 - 36	33 - 40	33 - 40	35 - 42	35 - 42	38 - 45	38 - 45	42 - 49	42 - 49	43 - 51
cB	18 - 20	18 - 20	20 - 22	20 - 22	22 - 24	22 - 24	24 - 26	24 - 26	26 - 29	26 - 29	29 - 32	29 - 32

Length in cm					
short	-	short	56 - 64	short	-
normal	35 - 40	normal	65 - 74	normal	65 - 74
long	41 - 45	long	75 - 84	long	75 - 84

c = circumference - l = length

COTTON

For the highest functional requirements of the skin

- High pressure stability
- For women and men with sensitive skin
- Excellent thermoregulation
- Very soft against the skin
- Optimal mix of fibres means that the stocking slides on and off easily
- Available in Closed and Open Toe
- Sizes available from XXS - XL PLUS

RAL CI 1 : 18 - 21 mmHg (light compression)
 RAL CI 2 : 22 - 32 mmHg (medium compression)
 RAL CI 3 : 33 - 46 mmHg (strong compression)

- | | | |
|--|--|--|
| COTTON CI 1
Maternity Tights | COTTON CI 2
Maternity Tights | COTTON CI 3
Maternity Tights |
| ■ Nature | ■ Nature | ■ Nature |
| ■ Black | ■ Black | ■ Black |



Circumference in cm	xx-small	xx-small PLUS	x-small	x-small PLUS	small	small PLUS	medium	medium PLUS	large	large PLUS	x-large	x-large PLUS
cT	60 - 75	65 - 80	65 - 80	75 - 90	75 - 90	85 - 100	85 - 100	95 - 110	95 - 110	105 - 120	105 - 120	115 - 130
cH	75 - 90	80 - 95	80 - 95	85 - 100	85 - 100	95 - 110	95 - 110	105 - 120	105 - 120	115 - 130	115 - 130	125 - 140
cG	42 - 54	47 - 59	47 - 59	52 - 64	52 - 64	56 - 68	56 - 68	61 - 74	61 - 74	66 - 79	66 - 79	76 - 85
cG Grip top	42 - 52	47 - 56	47 - 56	52 - 61	52 - 61	56 - 65	56 - 65	61 - 71	61 - 71	66 - 76	66 - 76	73 - 85
cC	25 - 32	29 - 36	29 - 36	33 - 40	33 - 40	35 - 42	35 - 42	38 - 45	38 - 45	42 - 49	42 - 49	43 - 51
cB	18 - 20	18 - 20	20 - 22	20 - 22	22 - 24	22 - 24	24 - 26	24 - 26	26 - 29	26 - 29	29 - 32	29 - 32

Length in cm					
short	31 - 34	short	56 - 64	short	56 - 64
normal	35 - 40	normal	65 - 74	normal	65 - 74
long	41 - 45	long	75 - 84	long	75 - 84
	lD		lG		lG

c = circumference - l = length

TRADITIONAL 500

For the highest medical standards

- High pressure stability
- Robust and durable
- Comfortable
- Only available in Open Toe and Beige colour
- Sizes available from XS - XL PLUS

RAL CI 2 : 22 - 32 mmHg (medium compression)

RAL CI 3 : 33 - 46 mmHg (strong compression)

RAL CI 4 : > 49 mmHg (very strong compression)

503 (CI.2)


Half thigh with grip top only available in small to large plus. Tights, maternity tights & tights for men only available in small plus, medium plus & large plus

504 (CI.3)

Half thigh with grip top only available in small to large plus


505 (CI.4)

Calf, half thigh, thigh only in plus sizes

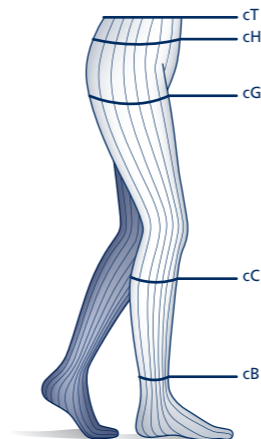

Circumference in cm	x-small*	x-small PLUS	small	small PLUS	medium	medium PLUS	large	large PLUS	x-large*	x-large PLUS*
cT			up to 80	up to 80	up to 90	up to 90	up to 100	up to 100		
cH			up to 110	up to 110	up to 120	up to 120	up to 130	up to 130		
cG	40 - 50	42 - 52	42 - 52	50 - 60	50 - 60	58 - 68	58 - 68	66 - 76	61 - 71	69 - 79
cC	28 - 31	30 - 33	31 - 36	35 - 40	34 - 39	39 - 44	37 - 42	43 - 48	40 - 45	47 - 52
cB	17 - 19	17 - 19	19 - 23	19 - 23	23 - 26	23 - 26	26 - 29	26 - 29	29 - 32	29 - 32

* only selected models

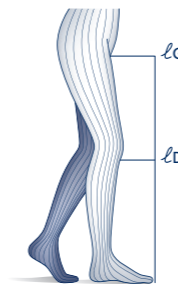

Length in cm					
short	33 - 37	short	61 - 70	short	61 - 70
long	38 - 44	long	71 - 80	long	71 - 80
	lD		lG		lG

c = circumference - l = length

Women

Women

ATTRAIT

- Contemporary striped pattern
- Elegant yet comfortable
- Soft against the skin
- Available in closed toe
- Sizes available from S - XL

AFNOR CI 2 : 15 - 20 mmHg (medium compression)

ATTRAIT

Calf

■ Black


ATTRAIT

Thigh with grip top

■ Black




	ANKLE b	CALF c	RETOUCH GRIP TOP	THIGH m
S	19 to 22	32 to 38	40	45 to 57
M	22 to 24	35 to 41	42	48 to 60
L	24 to 26	38 to 44	44	51 to 63
XL	26 to 29	41 to 47	46	54 to 66

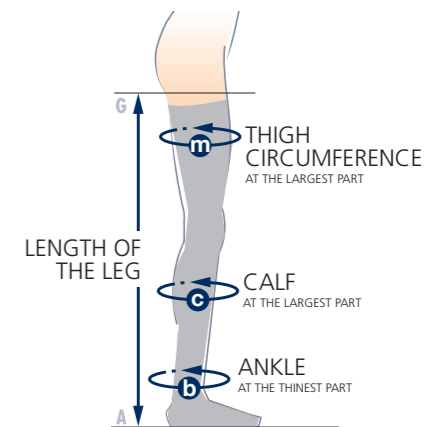

HEIGHT

	A-D	A-G	SHOE SIZES
NORMAL	≤ 40	≤ 75	36/40 - 3.5/6.5
LONG	≥ 40	≥ 75	37/41 - 4/7.5

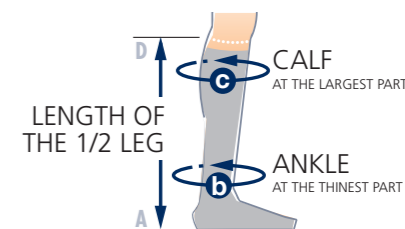

AUDACE

- Stylish and feminine
- Contemporary diamond pattern
- Comfortable and soft to touch
- Available in closed toe
- Sizes available from S - XL

AFNOR CI 2 : 15 - 20 mmHg (medium compression)

AUDACE

Calf

- Champagne
- Pearl
- Chocolate
- Black

AUDACE

Thigh with grip top

- Champagne
- Chocolate
- Black

AUDACE

Tights

- Black





	ANKLE b	CALF c	RETOUCH GRIP TOP	THIGH m
S	19 to 22	32 to 38	40	45 to 57
M	22 to 24	35 to 41	42	48 to 60
L	24 to 26	38 to 44	44	51 to 63
XL	26 to 29	41 to 47	46	54 to 66

	HEIGHT		SHOE SIZES
	A-D	A-G	
NORMAL	≤ 40	≤ 75	36/40 - 3.5/6.5
LONG	≥ 40	≥ 75	37/41 - 4/7.5

HIP CIRCUMFERENCE

TIGHTS
AUDACE

S	≤ 105
M	≤ 115
L	≤ 125
XL	≤ 125

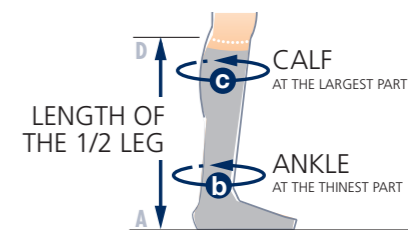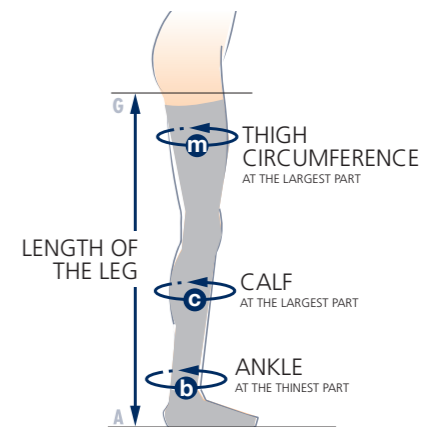

BAMBOU

- Natural fibre
- Easy to put on and take off
- Ideal for relaxation and leisure activities
- Available in closed toe
- Sizes available from S - XL

AFNOR CI 2 : 15 - 20 mmHg (medium compression)

BAMBOU


Calf

- Pale Cream
- Pale Peppermint
- Pale Rose
- Black



	ANKLE b	CALF c
S	19 to 22	32 to 38
M	22 to 24	35 to 41
L	24 to 26	38 to 44
XL	26 to 29	41 to 47

	HEIGHT		SHOE SIZES
	A-D	A-G	
NORMAL	≤ 40	≤ 75	36/40 - 3.5/6.5
LONG	≥ 40	≥ 75	37/41 - 4/7.5

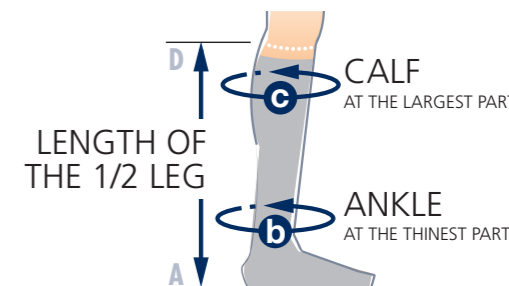

COTON

- Ideal for any season
- Helps to regulate skin temperature
- Very soft against the skin
- Available in closed toe
- Sizes available from S - XL

AFNOR CI 2 : 15 - 20 mmHg (medium compression)


COTON

Tights

■ Black



	ANKLE b	CALF c	RETOUCH GRIP TOP	THIGH m
S	19 to 22	32 to 36	40	45 to 51
M	22 to 24	35 to 39	42	48 to 54
L	24 to 26	38 to 42	44	51 to 57
XL	26 to 29	41 to 45	46	54 to 60

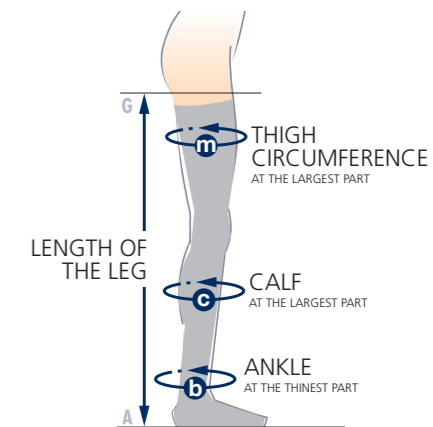

HIP CIRCUMFERENCE

TIGHTS
COTON

S	≤ 105
M	≤ 115
L	≤ 125
XL	≤ 125

HEIGHT

	A-D	A-G	SHOE SIZES
NORMAL	≤ 40	≤ 75	36/40 - 3.5/6.5
LONG	≥ 40	≥ 75	37/41 - 4/7.5

DIAPHANE

- Elegant and sheer
- Fashionable for all seasons
- Comfortable and soft to touch
- Sizes available from XS - XXL

AFNOR CI 2 : 15 - 20 mmHg (medium compression)



DIAPHANE

Calf

- Spice
- Amber
- Cinnamon
- Black OT*
- Natural OT*
- Dune OT*

DIAPHANE

Thigh with grip top

- Spice
- Amber
- Cinnamon
- Black OT*
- Natural OT*
- Dune OT*

DIAPHANE

Tights

- Spice
- Amber
- Cinnamon

DIAPHANE


Maternity Tights

- Black OT*
- Natural OT*

*Open Toe

	ANKLE b	CALF c	RETOUCH GRIP TOP 37	THIGH m
XS	17 to 19	29 to 37	37	42 to 62
S	19 to 22	32 to 40	40	45 to 65
M	22 to 24	35 to 43	42	48 to 68
L	24 to 26	38 to 46	44	51 to 72
XL	26 to 29	41 to 49	46	54 to 75
XXL	29 to 32	44 to 52	48	57 to 78


	HEIGHT		SHOE SIZES
	A-D	A-G	
NORMAL	≤ 40	≤ 75	36/40 - 3.5/6.5
LONG	≥ 40	≥ 75	37/41 - 4/7.5

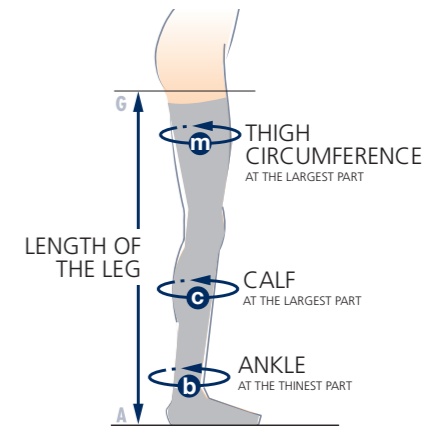

HIP CIRCUMFERENCE

TIGHTS
DIAPHANE

S	≤ 105
M	≤ 115
L	≤ 125
XL	≤ 125

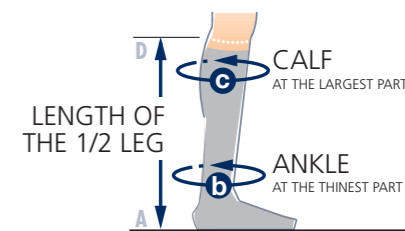

INTRIGUE

- Innovative large diamond design
- Extremely stylish
- Comfortable and soft
- Available in closed toe
- Sizes available from S - XL

AFNOR CI 2 : 15 - 20 mmHg (medium compression)

INTRIGUE

Thigh with grip top

■	Purple
■	Brown
■	Ink
■	Black

INTRIGUE

Tights

■	Purple
■	Brown
■	Ink
■	Black



	ANKLE b	CALF c	RETOUCH GRIP TOP	THIGH m
S	19 to 22	32 to 38	40	45 to 58
M	22 to 24	35 to 41	42	48 to 61
L	21 to 26	38 to 44	44	51 to 65
XL	26 to 29	41 to 47	46	54 to 68


HEIGHT

	A-D	A-G	SHOE SIZES
NORMAL	≤ 40	≤ 75	36/40 - 3.5/6.5
LONG	≥ 40	≥ 75	37/41 - 4/7.5

HIP CIRCUMFERENCE

TIGHTS
INTRIGUE

S	≤ 105
M	≤ 115
L	≤ 125
XL	≤ 125

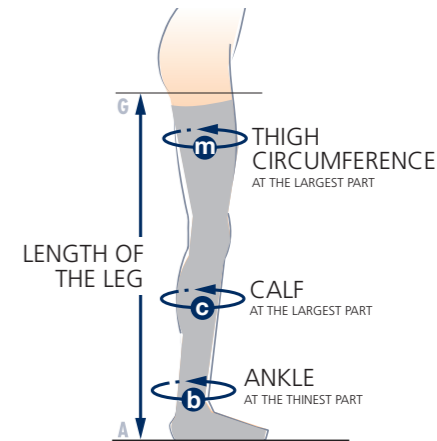

MAGIC

For the highest aesthetic standards

- Fine and elegant
- Semi-sheer finish
- Provides optimum support for the legs
- Available in Closed and Open Toe
- Sizes available from XXS - XL PLUS

RAL CI 1 : 18 - 21 mmHg (light compression)
 RAL CI 2 : 22 - 32 mmHg (medium compression)

MAGIC CI 1



- Calf
- Thigh with grip top
- Tights
- Maternity tights **

- Skin**
- Black**
- Caramel
- Savannah
- Brown
- Grey

MAGIC CI 2

- Calf
- Thigh with grip top
- Tights
- Maternity tights **

- Skin**
- Black**
- Caramel
- Savannah
- Brown
- Grey

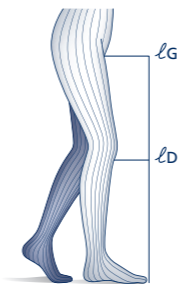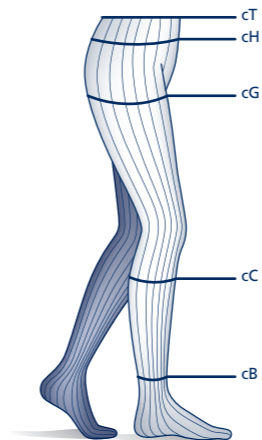

Circumference in cm	xx-small	xx-small PLUS	x-small	x-small PLUS	small	small PLUS	medium	medium PLUS	large	large PLUS	x-large	x-large PLUS
cT	60 - 75	65 - 80	65 - 80	75 - 90	75 - 90	85 - 100	85 - 100	95 - 110	95 - 110	105 - 120	105 - 120	115 - 130
cH	75 - 90	80 - 95	80 - 95	85 - 100	85 - 100	95 - 110	95 - 110	105 - 120	105 - 120	115 - 130	115 - 130	125 - 140
cG	42 - 54	47 - 59	47 - 59	52 - 64	52 - 64	56 - 68	56 - 68	61 - 74	61 - 74	66 - 79	66 - 79	76 - 85
cG Grip top	42 - 52	47 - 56	47 - 56	52 - 61	52 - 61	56 - 65	56 - 65	61 - 71	61 - 71	66 - 76	66 - 76	73 - 85
cC	25 - 32	29 - 36	29 - 36	33 - 40	33 - 40	35 - 42	35 - 42	38 - 45	38 - 45	42 - 49	42 - 49	43 - 51
cB	18 - 20	18 - 20	20 - 22	20 - 22	22 - 24	22 - 24	24 - 26	24 - 26	26 - 29	26 - 29	29 - 32	29 - 32

Length in cm					
short	-	short	56 - 64	short	-
normal	35 - 40	normal	65 - 74	normal	65 - 74
long	41 - 45	long	75 - 84	long	75 - 84

c = circumference - l = length

OPALIS

- Opaque colour range
- Soft on the skin
- Easy to get on and off
- Available in closed toe
- Sizes available from XS - XXL

AFNOR CI 2 : 15 - 20 mmHg (medium compression)

OPALIS

Tights

- Flesh
- Charcoal
- Black

OPALIS

Maternity tights

- Black




	ANKLE b	CALF c	RETOUCH GRIP TOP	THIGH m
XS	17 to 19	29 to 35	37	42 to 59
S	19 to 22	32 to 38	40	45 to 62
M	22 to 24	35 to 41	42	48 to 65
L	24 to 26	38 to 44	44	51 to 68
XL	26 to 29	41 to 47	46	54 to 71
XXL	29 to 32	44 to 50	48	57 to 74

	HEIGHT		
	A-D	A-G	SHOE SIZES
NORMAL	≤ 40	≤ 75	36/40 - 3.5/6.5
LONG	≥ 40	≥ 75	37/41 - 4/7.5

HIP CIRCUMFERENCE

TIGHTS
OPALIS

S	≤ 105
M	≤ 115
L	≤ 125
XL	≤ 125

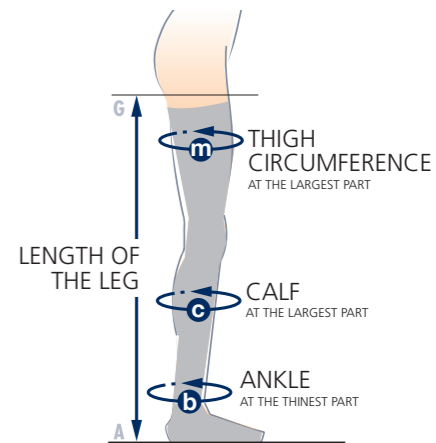

SOYANCE

- The first medical compression sock that contains silk
- Light and luxurious
- Feminine and stylish
- Available in closed toe
- Sizes available from S - XXL

AFNOR CI 2 : 15 - 20 mmHg (medium compression)

SOYANCE

Calf

- Mocha
- Black




	ANKLE b	CALF c
S	19 to 22	32 to 38
M	22 to 24	35 to 41
L	24 to 26	38 to 44
XL	26 to 29	41 to 47
XXL*	29 to 32	44 to 50

* Available in black only

HEIGHT

	A-D	A-G	SHOE SIZES
NORMAL	≤ 40	≤ 75	36/40 - 3.5/6.5
LONG	≥ 40	≥ 75	37/41 - 4/7.5

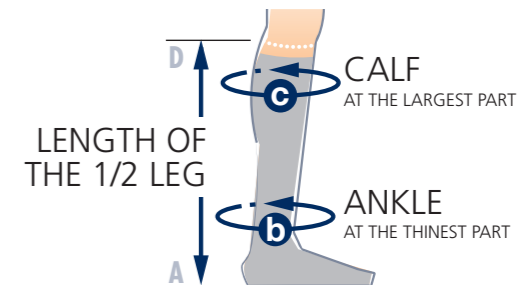

VERTIGE

- Elegant pattern
- Very light against the skin
- Comfortable and stylish
- Available in closed toe
- Sizes available from S - XL

AFNOR CI 2 : 15 - 20 mmHg (medium compression)

VERTIGE
Thigh with grip top
■ Black

VERTIGE
Tights
■ Black




	ANKLE b	CALF c	RETOUCH GRIP TOP	THIGH m
S	19 to 22	32 to 38	40	45 to 58
M	22 to 24	35 to 41	42	48 to 61
L	24 to 26	38 to 44	44	51 to 65
XL	26 to 29	41 to 47	46	54 to 68

HEIGHT			
	A-D	A-G	SHOE SIZES
NORMAL	≤ 40	≤ 75	36/40 - 3.5/6.5
LONG	≥ 40	≥ 75	37/41 - 4/7.5

HIP CIRCUMFERENCE

TIGHTS
VERTIGE

S	≤ 105
M	≤ 115
L	≤ 125
XL	≤ 125

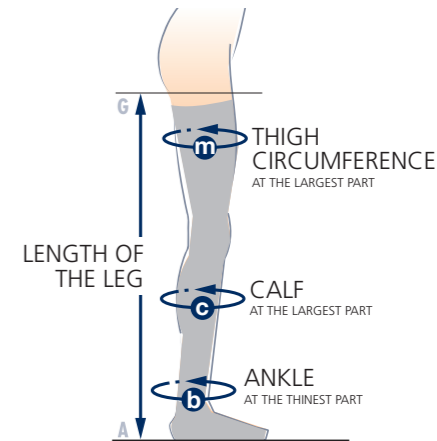

Men

Men

BAMBOU

- Natural fibre
- Easy to put on and take off
- Ideal for relaxation and leisure activities
- Available in closed toe
- Sizes available from S - XXL

AFNOR CI 2 : 15 - 20 mmHg (medium compression)

BAMBOU


Calf

- Pebble Grey
- Pacific Blue
- Black



	ANKLE b	CALF c
S	19 to 22	32 to 39
M	22 to 24	35 to 42
L	24 to 26	38 to 45
XL	26 to 29	41 to 48
XXL	29 to 32	44 to 51

	HEIGHT		SHOE SIZES
	A-D	A-G	
NORMAL	≤ 42	≤ 79	40/43 - 6/9
LONG	≥ 42	≥ 79	43/46 - 9/11

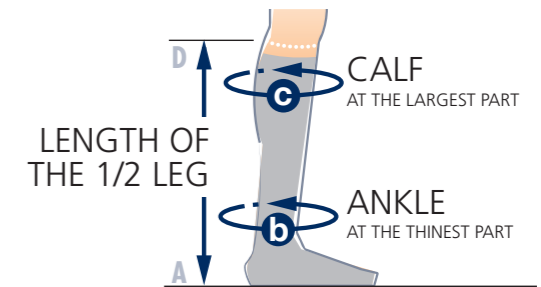

EXPERT

- Specifically shaped to provide comfort while wearing
- Fashionable for all seasons
- Comfortable and soft to touch
- Available in closed toe
- Sizes available from S - XXL

AFNOR CI 3 : 20 - 36 mmHg (STRONG compression)



EXPERT

Calf

■ Chrome



EXPERT

Thigh with grip top

■ Chrome

	ANKLE b	CALF c	RETOUCH GRIP TOP	THIGH m
S	19 to 22	32 to 39	40	46 to 51
M	22 to 24	35 to 39	42	48 to 54
L	24 to 26	38 to 42	44	51 to 57
XL	26 to 29	41 to 45	46	54 to 60
XXL	29 to 32	44 to 48	48	57 to 63

	HEIGHT		SHOE SIZES
	A-D	A-G	
NORMAL	≤ 42	≤ 79	40/43 - 6/9
LONG	≥ 42	≥ 79	43/46 - 9/11

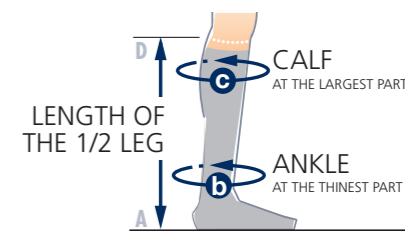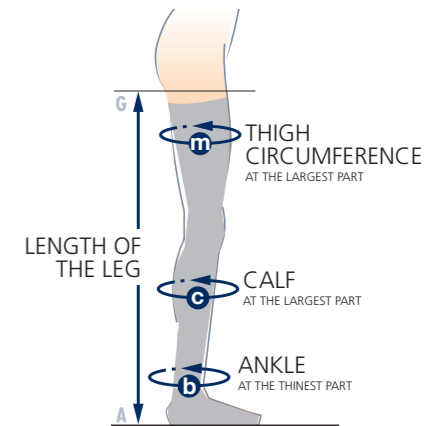

GRAPHIK

- Modern striped stylish design
- Easy to put on and to take off
- High-tech material helps to keep skin dry
- Available in closed toe
- Sizes available from S - XXL

AFNOR CI 2 : 15 - 20 mmHg (medium compression)



GRAPHIK

Calf


- Slate
- Cobalt
- Brown

	ANKLE b	CALF c
S	19 to 22	32 to 39
M	22 to 24	35 to 42
L	24 to 26	38 to 45
XL	26 to 29	41 to 48
XXL*	29 to 32	44 to 51

*Only available in Slate

HEIGHT

	A-D	A-G	SHOE SIZES
NORMAL	≤ 42	≤ 79	40/43 - 6/9
LONG	≥ 42	≥ 79	43/46 - 9/11

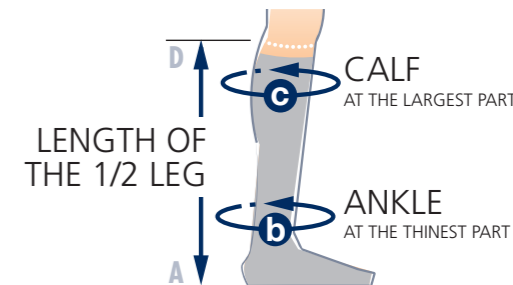

INITIAL

- Wicks away moisture from the skin
- Soft but durable fabric
- Offers all day comfort
- Available in closed toe
- Sizes available from S - XXL

AFNOR CI 2 : 15 - 20 mmHg (medium compression)

INITIAL

Calf

- Linen
- Nickel
- Steel
- Black




	ANKLE b	CALF c
S	19 to 22	32 to 39
M	22 to 24	35 to 42
L	24 to 26	38 to 45
XL	26 to 29	41 to 48
XXL*	29 to 32	44 to 51

*Only available in Linen, Nickel and Black

HEIGHT

	A-D	SHOE SIZES
NORMAL	≤ 42	40/43 - 6/9
LONG	≥ 42	43/46 - 9/11

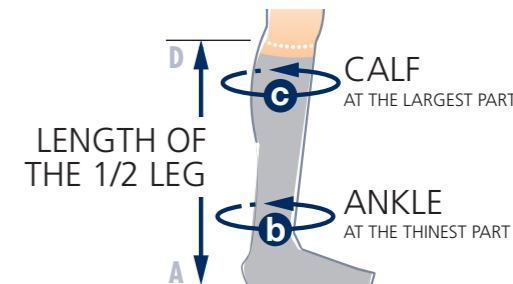

URBAN

- A ribbed sock look
- Ideal for leisure and sport activities
- High-tech material that wicks moisture
- Available in closed toe
- Sizes available from S - XXL

AFNOR CI 2 : 15 - 20 mmHg (medium compression)



URBAN

Calf

- Chrome
- Carbon
- Black *

URBAN

Thigh with grip top



- Chrome
- Black *

* Available in Open toe only

	ANKLE b	CALF c	RETOUCH GRIP TOP	THIGH m
S	19 to 22	32 to 39	40	46 to 58
M	22 to 24	35 to 42	42	50 to 62
L	24 to 26	38 to 45	44	54 to 66
XL	26 to 29	41 to 48	46	58 to 70
XXL	29 to 32	44 to 51	48	62 to 74

HEIGHT

	A-D	A-G	SHOE SIZES
NORMAL	≤ 42	≤ 79	40/43 - 6/9
LONG	≥ 42	≥ 79	43/46 - 9/11

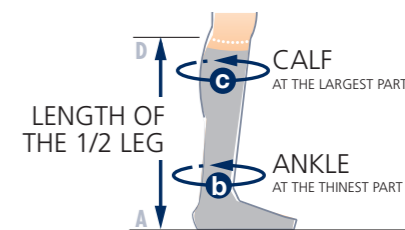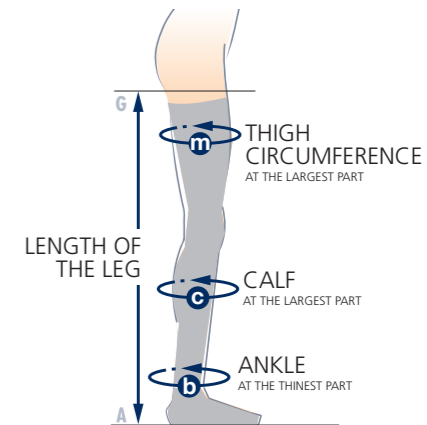

WOOL

- Ideal for any season
- Helps to regulate skin temperature
- Very soft against the skin
- Available in closed toe
- Sizes available from S - XL

AFNOR CI2: 15-20 mmHg (medium compression)


WOOL

- Black



	ANKLE b	CALF c
S	19 to 22	32 to 39
M	22 to 24	35 to 42
L	24 to 26	38 to 45
XL	26 to 29	41 to 48

	HEIGHT		
	A-D	A-G	SHOE SIZES
NORMAL	≤ 42	≤ 79	40/43 - 6/9
LONG	≥ 42	≥ 79	43/46 - 9/11

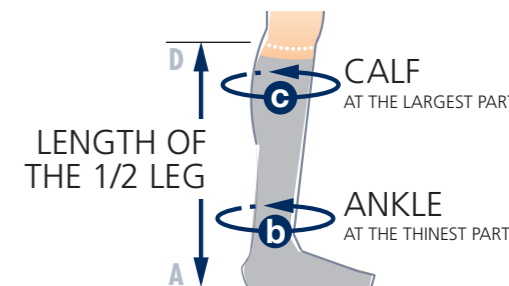

More

More

Accessories

	Product code
Sigvaris FIX glue	12972
Helps to keep medical compression hosiery in place	
Cleaning Wipes	20473
Enhance the performance of grip tops on compression hosiery	
Washing solution	16276
Helps compression hosiery to maintain their softness and durability	
Rubber Gloves*	
Small	12973
Medium	12975
Large	12977
Extra Large	17239
Textile Gloves*	
Extra Small	87276
Small	87277
Medium	87278
Large	87279
Extra Large	87280
Stocking Aids*	
Round	109000
Oval	109001
Doff N' Donner Cone	63458

* Ideal products to aid people with donning and doffing of medical compression hosiery.



Glue



Washing Solution



Wipes



Textile Gloves



Rubber Gloves



Doff N' Donner Cone



Stocking Aids

Lymphoedema Gloves

Concept

Palm Measurement	Wrist measurement
16	17.0
17	17.5
18	18.0
19	18.5
20	19.0
21	19.5
22	20.0
23	20.5
24	21.0
25	21.5
26	22.0
27	22.5
28	23.0
29	23.5



Class 2 (23 - 32 mmHg)
Micro-massage with each movement,
Available in 14 sizes,
seamless hand finished for comfort Also available in brown and black (by special order)

Traditional natural rubber quality mitten

Size	cA
Small	16 - 19
Medium	19 - 22
Large	22 - 25



Truefit



Class 2 (23 - 32 mmHg), Skin friendly, light, soft fabric, Elasticity from 100-250% Flat Knit design

Size	Palm	Wrist measurement		
1	16/17 cms	14	15	16
2	18/19 cms	15	16	17
3	20/21 cms	16	17	18
4	22/23 cms	17	18	19
5	24/25 cms	18	19	20
6	26/28 cms	19	20	21

Lymphoedema Glove (ambidextrous)



Class 1 (18 - 21 mmHg)
Light fabric and comfortable.

Size	cA
Small	16 - 19
Medium	19 - 22
Large	22 - 25

TRAVENO Travel socks

- Intensive graduated support
- Reduced swelling of the legs
- Breathable and easy-care
- Ideal for long journeys
- Available for women and men

15 - 18 mmHg

TRAVENO

Calf

- Anthracite
- Marine
- Sand
- Black




UK SHOE SIZE PRODUCT SIZE

3.5 - 4.5	1
5 - 6	2
6.5 - 7.5	3
8 - 9	4
9.5 - 10.5	5
11 - 12	6

CHIPSLEEVE


- Mild graduated compression
- Open cell foam chips to help fibrotic areas
- Fabric features moisture wicking
- Odour control properties

15-25 mmHg

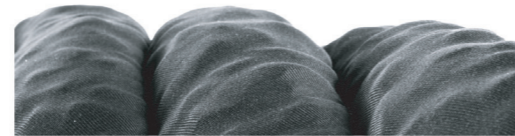


DIRECTIONAL CHANNELS

Channel formations facilitate efficient lymph drainage

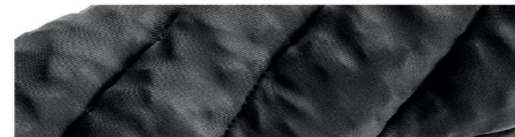

FOAM CHIPS

Open cell foam chips massage superficial lymphatics

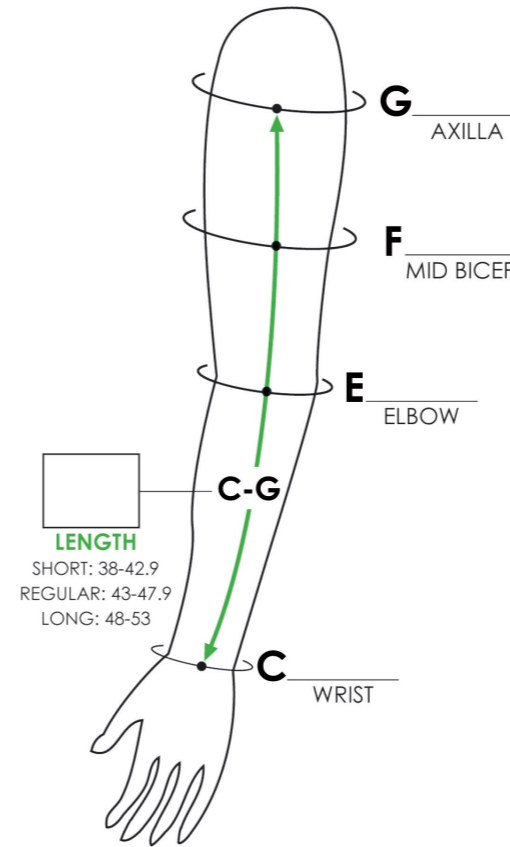

CIRCULAR MEASUREMENTS

Size	Small	Medium	Large	XLarge	XXLarge
G	23 - 32	28 - 37	33 - 43	39 - 49	44 - 54
F	22 - 30	26 - 34	30 - 39	35 - 44	39 - 49
E	20 - 27	24 - 31	28 - 35	3 - 44	36 - 43
C	14 - 17	15 - 18	16 - 19	18 - 20	19 - 22

PRODUCT CODES

Length	Left side				
Short (38-42.9)	2631-AS-L	2632-AS-L	2633-AS-L	2634-AS-L	2635-AS-L
Regular (43-47.9)	2631-AR-L	2632-AR-L	2633-AR-L	2634-AR-L	2635-AR-L
Long (48-53)	2631-AL-L	2632-AL-L	2633-AL-L	2634-AL-L	2635-AL-L
Length	Right side				
Short (38-42.9)	2631-AS-R	2632-AS-R	2633-AS-R	2634-AS-R	2635-AS-R
Regular (43-47.9)	2631-AR-R	2632-AR-R	2633-AR-R	2634-AR-R	2635-AR-R
Long (48-53)	2631-AL-R	2632-AL-R	2633-AL-R	2634-AL-R	2635-AL-R

CHIPSLEEVE

- Black

SIGVARIS

SIGVARIS over the world

SIGVARIS Britain Ltd

For more information on any of our compression hosiery please call our dedicated customer care line on **01264 326 666**.

Or if you prefer you can fax us on **01264 369 950**

Or visit our website at **www.sigvaris.co.uk**

SIGVARIS BRITAIN LTD

1, Imperial Court, Magellan Close,
Andover, Hampshire SP10 5NT



SIGVARIS are the world leaders in the field of medical compression stockings, available in over 70 countries worldwide.

